

Vista Irrigation Regimen...



Why Contact Time is so Important:

Decreases need for retreatment:

- From bacteria left behind
(Bacteria not removed by instrumentation)
- From restoration leaks

Vista's 5-Step Irrigation Regimen:

Your **BEST** approach to disinfecting canals

- Recommended to yield best outcomes
- Optimizes the time spent on irrigation

Follow these 5 easy steps:

1. Chlor-XTRA™ 10-12 ml
2. EDTA 6 ml
3. Chlor-XTRA™ 10-12 ml
4. Rinse-N-Dry™ 6 ml
5. CHX-Plus™ 6-12 ml

NEW!

5-Step Irrigation Regimen Kit

- No mess . . . less time!
- All your endo solutions in one easy kit!
- Fill syringes right from the bottle with our convenient docking port cap.



Kit Includes:

- (2) bottles Chlor-XTRA™ (12 ml)
- (1) bottle EDTA (6 ml)
- (1) bottle Rinse-N-Dry™ (6 ml)
- (1) bottle CHX-Plus™ (12 ml)
- (5) color coded syringes



(4 Kits / Box)
\$60.00

To Improve Your Outcomes! Exclusively from Vista!

Improve outcomes with our recommended irrigation

Buy 4 Get 1 FREE!



Safety-Irrigator™

A simple negative pressure, one-hand use, simultaneous irrigation and evacuation device, which when used properly provides safe, easy-to-use, cost effective, apical irrigation and evacuation. **New, improved needle tip** can be cut to actual working length for controlled apical third placement!! Also features ratcheting plunger for controlled delivery of solutions.

20 Pack Safety-Irrigator™ . . . \$55.00



1 + 3 Chlor-XTRA™ (Multiple Patents Pending) Irrigation, debridement and disinfection of root canals during and after instrumentation.

This 6% Sodium Hypochlorite formula, enhanced with powerful wetting agents and propriety surface modifiers, penetrates into lateral canals and isthmuses by lowering the surface tension, thus yielding a much more powerful oxidizing agent. Over 2 times thinner than standard Sodium Hypochlorite and 2+ times more digestive.

16oz Bottle Chlor-XTRA™ . . . \$57.50

2 EDTA 17% solution For removal of the smear layer and dentin mud for maximum disinfection of dentinal tubules.

Tip: Remember to flush with enough EDTA because Sodium Hypochlorite will decrease effectiveness of EDTA.

16oz Bottle EDTA . . . \$45.70

4 Rinse-N-Dry™ 95% Ethyl Alcohol (Multiple Patents Pending)

Low surface tension penetrates lateral canals and isthmuses to improve drying / disinfecting properties. Use after smear layer removal with EDTA. Assures canal is clear of all Sodium Hypochlorite before final irrigation step with CHX-Plus™.

4oz Bottle Rinse-N-Dry™ . . . \$8.00

5 CHX-Plus™ (Multiple Patents Pending) Final irrigation step for long-lasting disinfection.

2% Chlorhexidine Gluconate with powerful wetting agents and propriety surface modifiers. Improves depth of penetration and kills bacteria 10X faster than normal 2% Chlorhexidine.

Caution: Never mix CHX with Sodium Hypochlorite because it creates a toxic precipitate.

16oz Bottle CHX-Plus™ . . . \$53.00

For other solution configurations, or our FREE General Products and Comparison catalogs, call or visit our website.

AVAILABLE THROUGH YOUR PREFERRED DEALER

www.vista-dental.com

© 2010 Vista Dental Products